EDGE^{EOM}

Maxillofacial and Oral Surgery

 EDGE SURGICAL SOLUTIONS

Registered Community Design

WHO WE ARE | QUI NOUS SOMMES | CHI SIAMO

THE COMPANY

EDGE SURGICAL SOLUTIONS is specialized in developing and manufacturing Multispecialty Examination and Surgery Chairs, Surgeon's Stools and Furniture.

EDGE SURGICAL SOLUTIONS is committed to the continuous research of innovative technical solutions for the Professionals; the range of equipment offered is based on and follows rigid guidelines: latest technology, perfect work-flow ergonomics, reliability, design and highly intuitive and easy to operate by the Professional.

The Company is directly involved through all development, manufacturing, assembly and quality control, shipment, installation and after-sale service phases thanks to its vast network of Distributors and Agents.

In the Company premises there is a permanent Show-Room and a Training Area for theory-practice sessions.

A well structured company organization that has enabled EDGE SURGICAL SOLUTIONS to grow at national and international level keeping the highest quality standards and constant innovation of its range of products.

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“ALLIEDGE SURGICAL SOLUTIONS Medical Equipment is designed, developed and manufactured in Italy”

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HOW WE WORK | COMMENT NOUS TRAVAILLONS | COME LAVORIAMO

R&D

EDGE SURGICAL SOLUTIONS R&D Team develops internally all the equipment and devices for the Professionals and Clinics, always combining modern and essential design to the latest and most innovative technical solutions.

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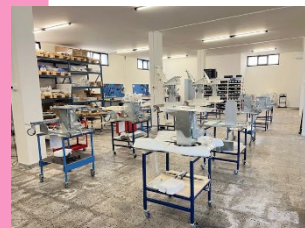
R&D
R&D
R&D

EDGE SURGICAL SOLUTIONS manufactures a wide and complete range of Surgical Chairs, Doctor Stools and Furniture. Every single production phase is planned and performed according to the most rigid quality control criteria.

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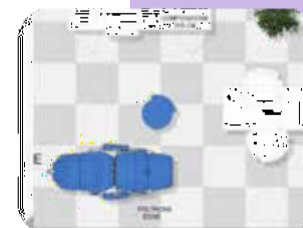
PRODUCTION
PRODUCTION
REALIZZAZIONE

EDGE SURGICAL SOLUTIONS will design your Ambulatory Surgery Center in order to allow for the best work-flow and ergonomics while guaranteeing maximum comfort to the Patient, according to the space availability and your different set up needs as a Professional.

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PROJECT
PROJET
PROGETTO

INSTALL
INSTALL
INSTALL

All EDGE SURGICAL SOLUTIONS equipment is "Plug-in & Work" easy to set up and ready to use with very few and easy installation steps. EDGE SURGICAL SOLUTIONS guarantees a highly professional installation service thanks to its vast network of Distributors and Agents, and a product specialist will train the Ambulatory staff on the correct use of the installed equipment.

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EDGE SURGICAL SOLUTIONS is committed to continuous training and information of all its Customers; the Company can rely on the cooperation with prominent experts in each medical discipline and often involved in training sessions on the specific equipment and different operating methods.

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EDGE SURGICAL SOLUTIONS considers Warranty a professional duty and mandatory towards all its Customers, reason being the Company extends guarantee of the equipment up to 2 Years on Electronic Parts and up to 5 Years on Mechanic Parts.

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All Customers can rely on a punctual and qualified after-sale assistance thanks to the in-house Service Department. You will be able to request a Preventive Maintenance and Courtesy Service to our Distributors and avoid any unexpected interruption of your professional activity.

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SERVICE
SERVICE APRES-VENTE
ASSISTENZA

AMBULATORY SURGERY | CHIRURGIE AMBULATOIRE | CHIRURGIA AMBULATORIALE

A Transformative Model for Modern Surgery and Procedures in ASC and Outpatient Rooms.

Ambulatory Surgery Centers—known as ASCs—are modern health care facilities focused on providing same-day surgical care, including diagnostic and preventive procedures.

ASCs have transformed the outpatient experience for millions of Patients around the World by providing them with a more convenient alternative to hospital-based outpatient procedures—and done so with a strong track record of quality care and positive patient outcomes while proving highly cost-effective.

Forty years ago, virtually all surgery was performed in hospitals; waits of weeks or months for an appointment were not uncommon, and

patients typically spent several days in the hospital and several weeks out of work in recovery. In many countries, surgery is still performed this way, but it is rapidly changing.

Physicians faced with frustrations like scheduling delays, limited operating room availability, slow operating room turnover times, and challenges in obtaining new equipment due to hospital budgets and policies, physicians were looking for a better way—and developed it in ASCs.

EDGE SURGICAL SOLUTIONS has fully committed to give a contribution to the development of technically advanced, elegant and cost-effective equipment that enables Physicians to perform Examinations and Procedures according to the time saving principle of rotation: prepare the Patient, perform Surgery and keep Patient for observation and recovery on the same Chair. When the Physician is

operating, staff can prepare the following Patient for surgery and at the same time the previous Patient is safely and comfortably recovering on the same chair. The Patient is never moved from the chair, access is easily granted thanks to the extremely low entry/exit height of the chair, and then by simply pressing the memory buttons the Patient will immediately reach operating position previously set and memorized by the Physician. The chair turns to an operating table, then to a recovery bed and back to examination chair by simply operating the wireless foot-control, and all of this performed at ease, with maximum comfort for the Patient and best ergonomics for the Physician.

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Modern Surgery and Procedures in ASC and Outpatient Rooms

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EDGE EOM

Maxillofacial and Oral Surgery Chair

It is EDGE SURGICAL SOLUTIONS Ultimate concept of Maxillofacial Examination and Treatment Chair dedicated to Ophthalmology, ENT and Oral Surgery; it is the result of innovative technical and engineering solutions that enable the Professionals to perform any examination and treatment in the outpatient and/or Ambulatory Surgery Centres.

It is the paramount tool that suits all clinics and private practices.

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EDGE EOM

EDGETM can be equipped with different accessories transforming the chair in the perfect working tool in Outpatient Surgery, Hair-Transplant FUT & FUE procedures, Bariatric and Surgical Podiatry, ENT, Dermatology and Cosmetic Surgery, Diagnostic, Ambulatory Surgery, General Examination and Minor Procedures.

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EDGE EOM-P EVO

EDGE EOM is conceived in a solid metal structure, the mobile base is equipped with 4 medical grade castors and rear lock and It is operated with 3 silent linear actuators and one telescopic column by a modern and multi-function wire-less foot control. The upholstery comprises 5 separate padded sections fully removable for easy cleaning and disinfection, made of high density undeformable foam and the cover is an abrasion-resistant, flame retardant and antimicrobial Skai available in 80 different colours and 8 textures.

EDGE EOM is designed for the comfort of the patient and the ergonomic of the operator, it reaches a minimum height of 53 cm (21" inches) and a maximum height of 93 cm (37" inches) allowing patients with medical conditions to easily access the chair from the front or the sides without any barrier, while allowing the operator to position the chair in a multiple range of heights and inclinations enabling the perfect operating position.

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EDGE EOM-PEVO

is equipped with 4 wheels run by an electronic brake system. can be locked and released easily and ergonomically by means of the key buttons situated on the back of the Chair. A wheel with directional lock allows the Professional to move the Chair over long distances with ease.

The motion sensor connected to the wheels allows the Chair to be automatically locked after 30 sec. of stop.

Is Equipped with a 5.0 Ah battery allowing it to operate the Chair autonomously, without connection to the power supply.

The battery allows 200 movements of the Chair. The charge indicator on the battery allows monitoring the charge level.

The battery recharges autonomously when the Chair is connected to the power supply.

PROVIDED IN THE STANDARD VERSION

- Total or directional lock (0/180°)
- Housing and wheel centre glass fibre- reinforced polyamide
- Wheel tread polyurethane 80 shore A, hydrolysis resistant
- Ball bearing in swivel head and double row ball bearing in wheel
- Wheel diameter 125 mm
- Load capacity 150 kg

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EDGE EOM-P

EDGE EOM can be equipped with different accessories transforming the chair in the perfect working tool in Outpatient Surgery, Hair-Transplant FUT & FUE procedures, Bariatric and Surgical Podiatry, ENT, Dermatology and Cosmetic Surgery, Diagnostic, Ambulatory and Minor Procedures.

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EDGE EOM is conceived in a solid metal structure, painted with epossidic powder and operated with 3 silent linear actuators and one telescopic column by a multifunction foot control. The upholstery comprises 4 separate padded sections fully detachable for easy cleaning and disinfection, made of high density undeformable foam and the cover is a bi-elastic, flame retardant and antimicrobial medical grade Skai available in 80 different colours and textures.

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EDGE EOM

The chair reaches a minimum height of 48 cm allowing patients with different medical conditions to easily access the chair from the front or the sides without any barrier.

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It can automatically reach the Operating Position thanks to the memorizable positions function activated by the foot control.

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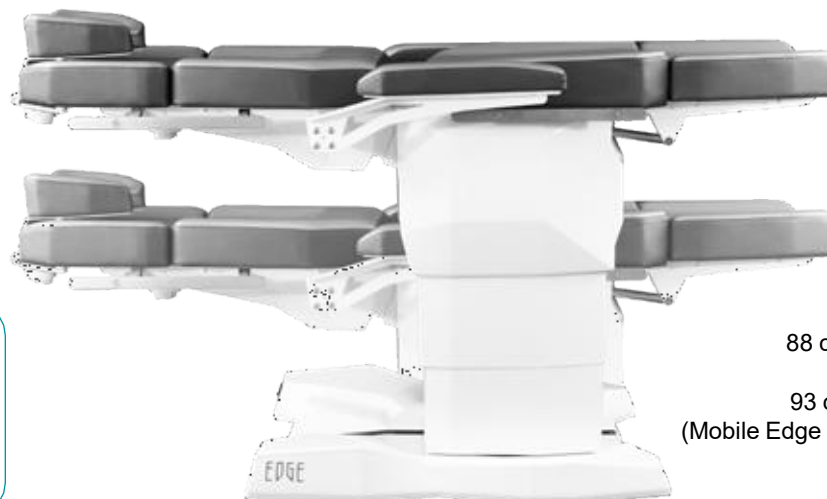
It is equipped with medical grade castors, two of them are pivoting while 2 fixed, and central brake system at rear, in order to provide maximum maneuverability.

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The chair can reach a perfect flat position at any height, for the maximum comfort of the Operator while performing a clinical examination or professional procedure. Maximum height 88 cm.

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Zero position of the chair.
The chair automatically reaches "0" position to facilitate the access and the exit of the Patient.

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It allows the operator to perform "Synchro" movements of the Chair enables the combined and synchronized movement of the back-rest and leg-rest.

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It can rely on a lifting capacity of 325 kg, one of the highest in its category.

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It reaches Trendelenburg position with 20° inclination.

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EDGE EOM

Head-rest is shaped and padded according to the requirements of different medical sectors and can all be completely removed when necessary.

The articulated head-rests are adjustable in 3 directions, can be locked firmly when reaching the correct position of the Patient.

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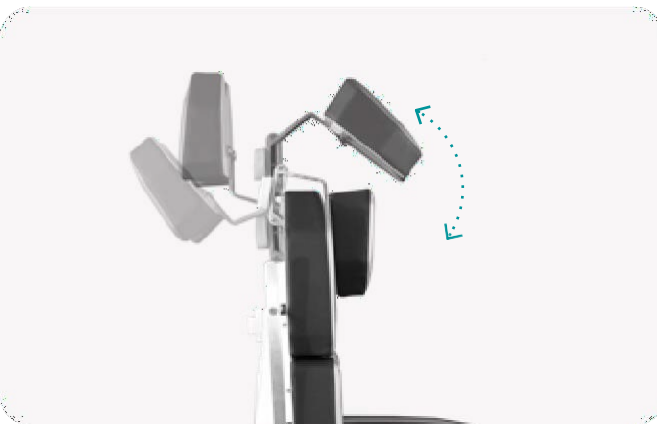
**OPTIONAL HEAD-REST
MORE INFO ON PAGE 18-19**

The headrest can be positioned in line with the back-rest upholstery. The sliding rod of the headrest allows a height excursion of 30 cm.

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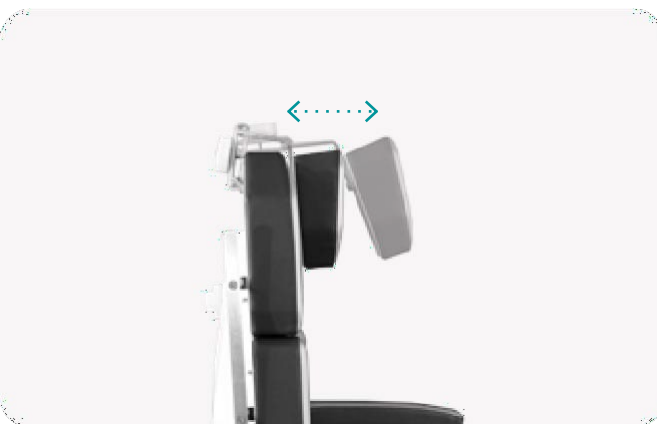
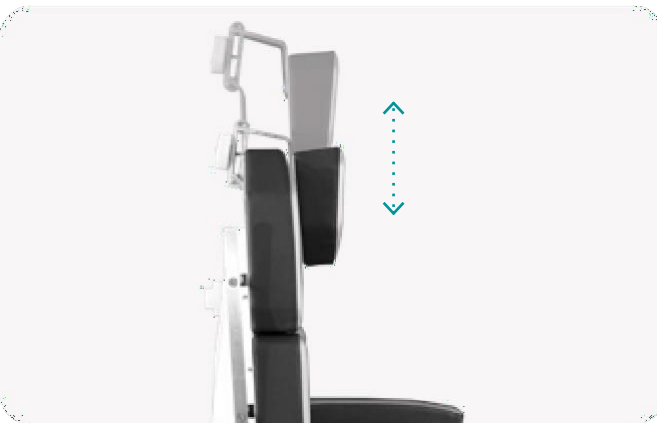
Arm-rest is shaped and padded according to the requirements of different medical sectors and can all be completely removed when necessary.

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Both padded arm-rest are independently collapsible, removable and mechanically combined with the seat/back-rest movements ensuring comfort for the patient in any position.

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EDGE EOM

Padded and shaped leg-rest.
PROVIDED IN THE STANDARD VERSION

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Leg-rest operated with a multifunction foot control by with a linear actuator.

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2 Stainless steel side rails mounted on the seat.
PROVIDED IN THE STANDARD VERSION

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Back-rest Side Rail, ideal for all standard clamps and accessories.
OPTIONAL

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OPTIONAL BACK-REST SIDE RAIL
MORE INFO ON PAGE 21

EDGE EOM

It is provided in the standard version with the structure in White color or Silver at choice.

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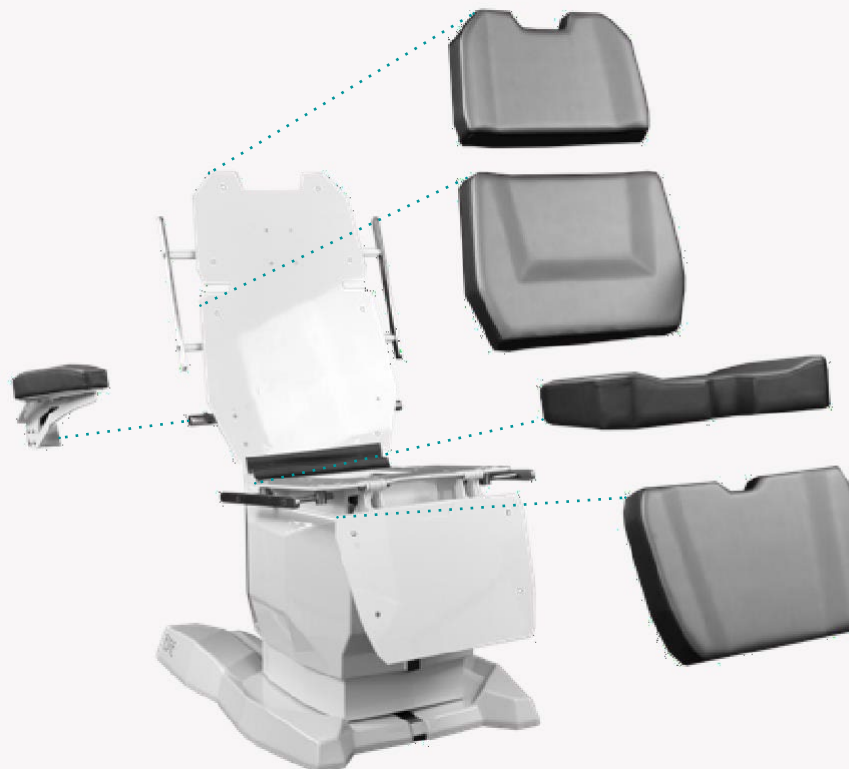
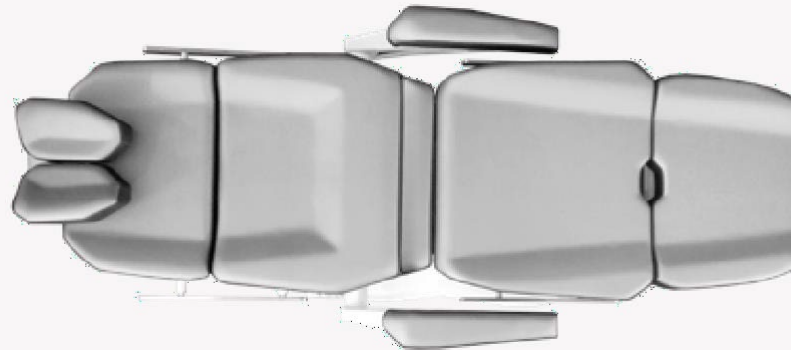
The upholstery comprises 4 separate padded sections fully detachable for easy cleaning and disinfection, made of high density undeformable foam and the cover is a bi-elastic, abrasion-resistant, flame retardant and antimicrobial medical grade Skai available in over 80 different colours and textures.

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EDGE EOM

It is operated by a wireless foot control with modern design and extremely intuitive. Easy to operate and clean, the foot control provides the Professional with an ergonomic and efficient working tool, useful to position the Patient comfortably and memorize different working or safety positions.

The foot control has 8 buttons to operate the 4 actuators and move the Chair sections; it is also provided with 2 memorizable positions buttons (M1,M2), the easy Patient Access-Exit position (M0), and the Sync button that enables leg-rest and back-rest simultaneous movement.

The foot control is also equipped with two buttons for the emergency positions of the Chair: the flat position and the Trendelenburg position.

The pedal innovative visual and acoustic signals communicate all safety, memorized movements and charging activity of the chair and it's foot control.

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Style Head-Rest. Comfortable support for the Patient, ideal for any general examination or minor procedure.

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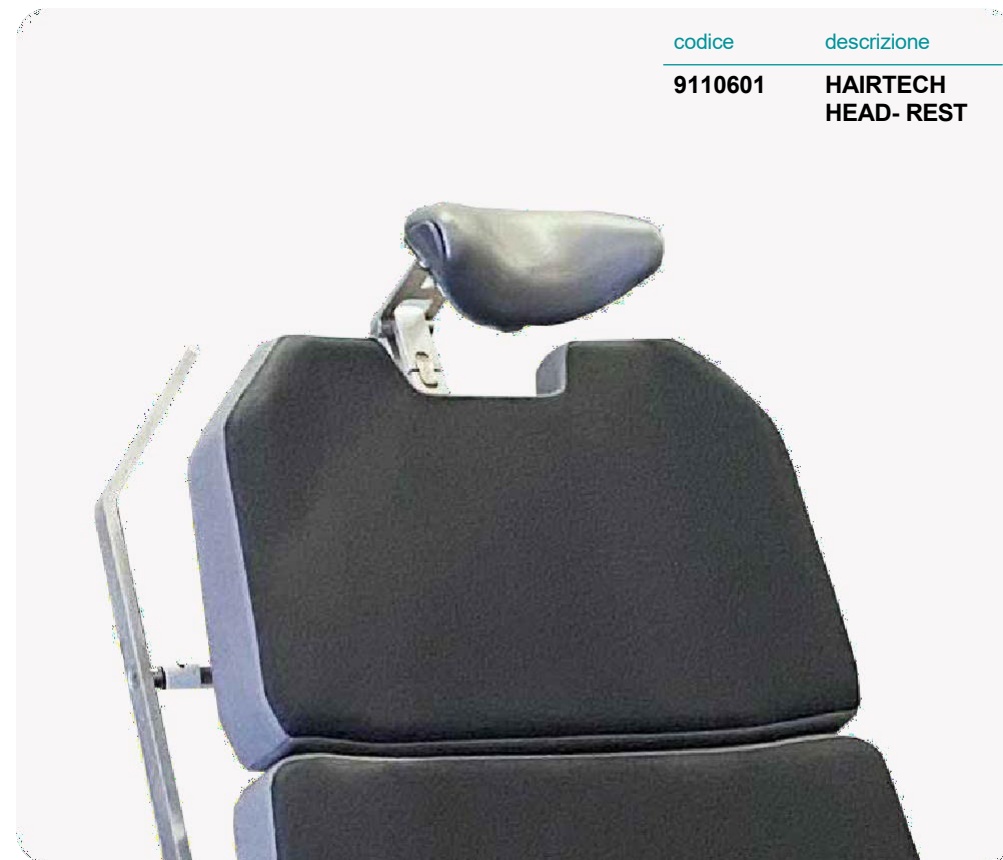
Ophthalmic and ENT Head-Rest. **PROVIDED IN THE STANDARD VERSION (EDGE EOM-P)**
OPTIONAL (EDGE EOM)

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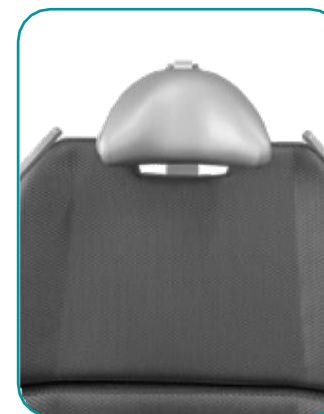
HairTech Head-Rest grants ideal cervical support to the Patient while guaranteeing the perfect ergonomics to the Doctor. **OPTIONAL**

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Triple Articulation Head-Rest for perfect positioning of the Patient's head in any required cranio-facial examination/surgery.

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OPZIONALE

Articulated joint available with Style headrest, Hair-Tech headrest and Ophthalmic headrest.

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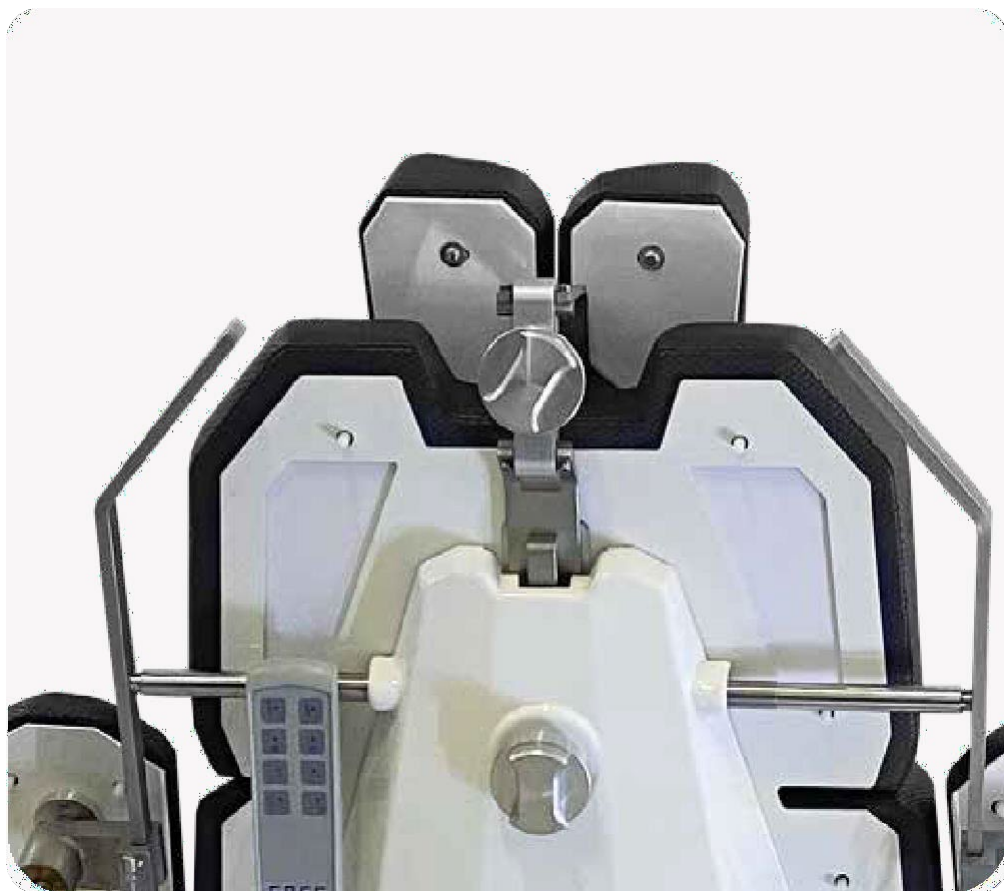
2 locking knobs in highly resistant and durable stainless steel, easy to operate and disinfect.
PROVIDED IN THE STANDARD VERSION

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Lower section of the back-rest without lumbar support. Alternative to the one with lumbar support.
OPTIONAL

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Back-rest Side Rail, ideal for all standard clamps and accessories.

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OPZIONALE

Side Support, it is ideal for patient stabilization and containment; it is collapsable and removable. Easy to clean and disinfect.

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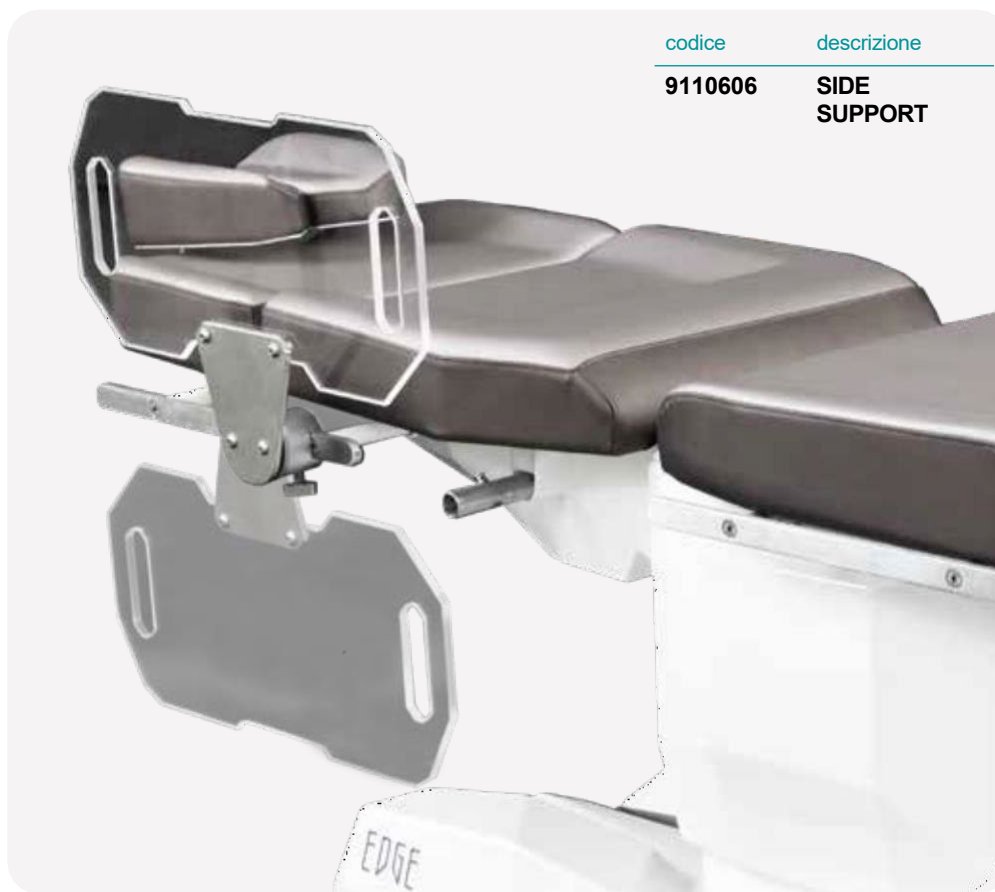
OPZIONALE

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I.V Arm-rest, it rotates 360° and is swiveling and tiltable by means of ball joint.

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Stainless steel tray support, extendable and height adjustable.

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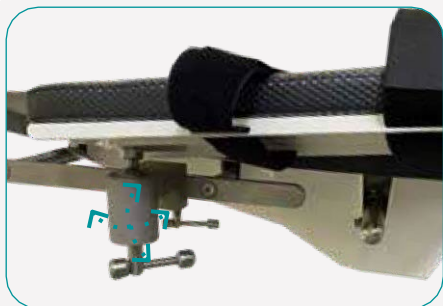
Articulated armrests on mechanical ball joint with removable upholstery and Velcro containing belt.

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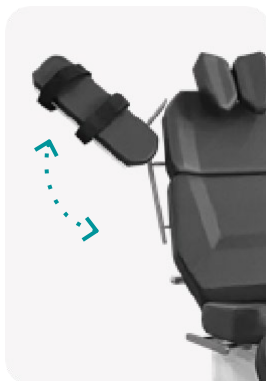
EN OPTION

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OPZIONALE

Lumbar Support Pillow guarantees maximum comfort to the patient in supine position.

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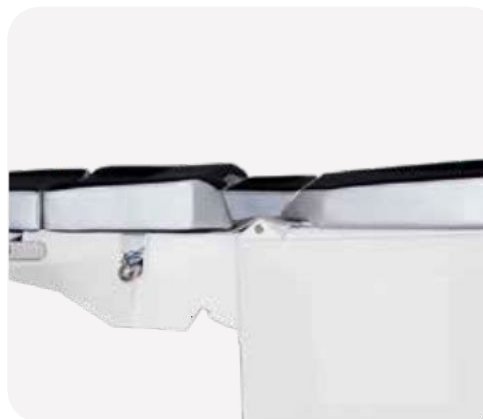
Paper Roll Support, adjustable for different roll sizes.

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Maniglie.

OPZIONALE

codice	descrizione
9110636	FLEXIBLE DRAPE SUPPORT WITH OXYGEN CONNECTION

Flexible Surgical Drape Support
with Oxygen Connection and Clamp
Included

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Flexible Surgical Drape Support with Clamp Included
OPTIONAL

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Body Velcro tape for Patient stabilization and containment.
OPTIONAL

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Belt with buckle for stabilization and containment of the Patient.
OPTIONAL

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Battery kit including two batteries (capacity 5Ah), with capacity indicator and recharger, allowing to operate the Chair without electrical supply connection. Replacing the battery with the spare one provided is very simple.

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Back up battery with 2Ah capacity allowing to operate the Chair in the event of a temporary power outage.

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OPZIONALE

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The Hand Control enables the required positioning of the Patient, by activating the linear actuators that control the position of each independent section (lifting column, seat, back-rest and foot-rest).

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codice	descrizione
9110631	RACK FOR MEDICAL ACCESSORIES (white structure)
9110631S	RACK FOR MEDICAL ACCESSORIES (silver structure)

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OPZIONALE

LED Examination Lamp, 100.000LUX at 50cm distance, 5.500°K, CRI 98. Through touch control panel it is possible to manage all the functions of the lamp.

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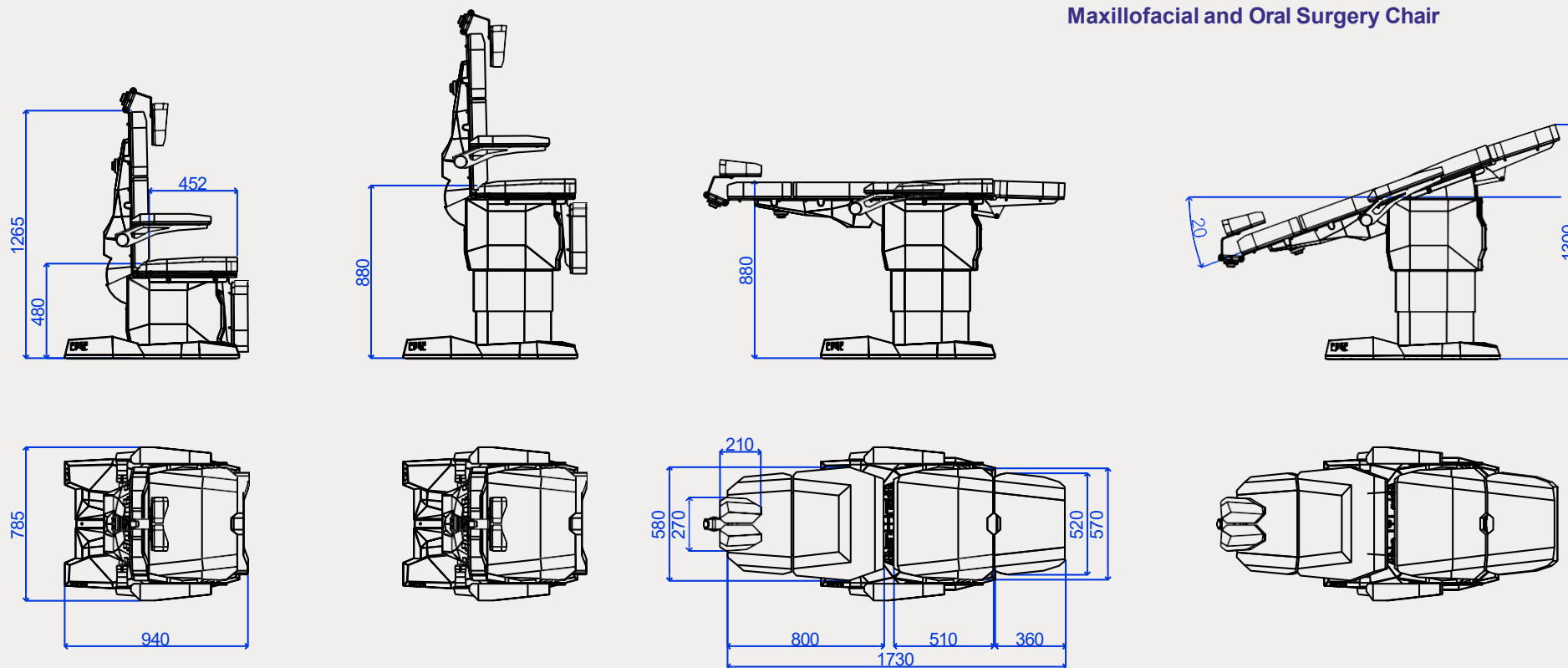
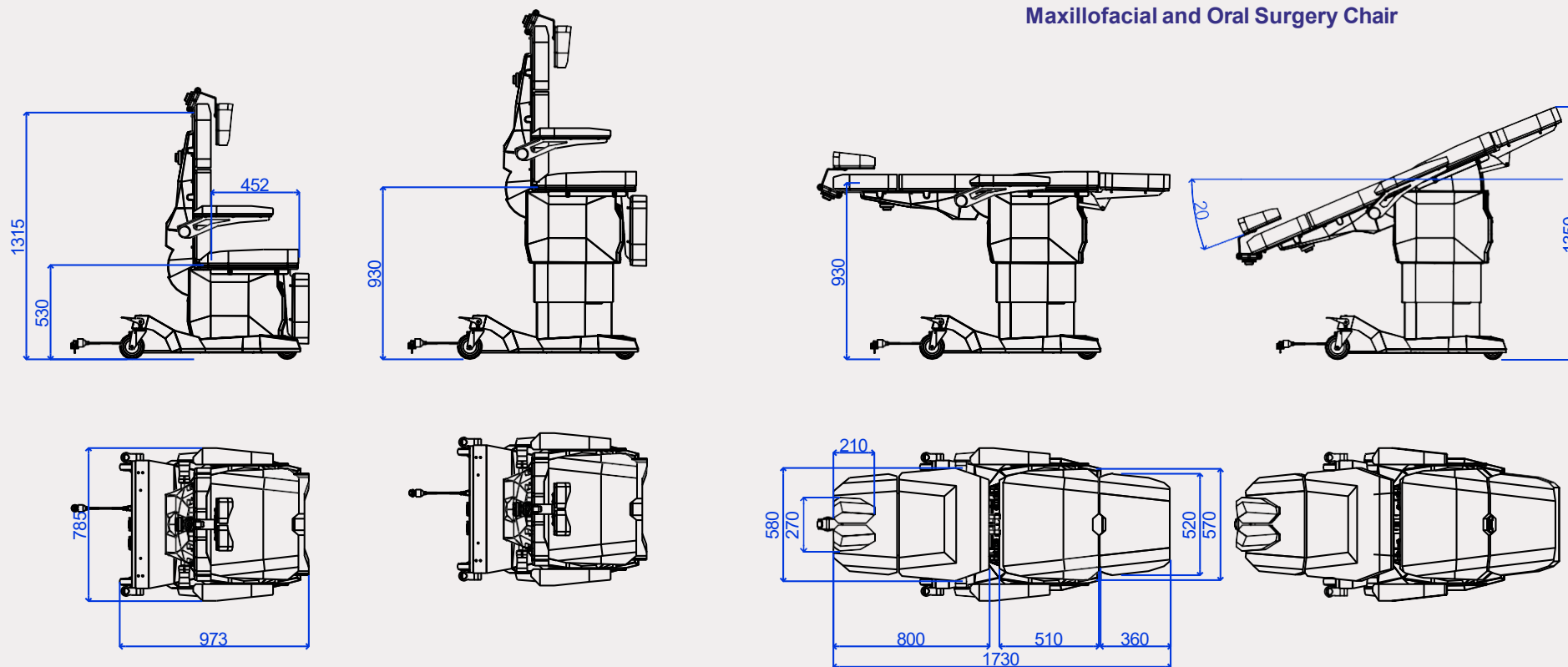
24 V

Absorption
Absorption
 Assorbimento

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The seat naturally adapts to the user's body by increasing the contact area and decreasing the pressure on it. Circular seat with visco-elastic foam in different densities and memory effect. Multifunction armrest/ back rest in high resistant polyurethane. Base in steel and polished chrome. 5 self braking wheels.
Adjustable height by gas piston. High quality skay cover.

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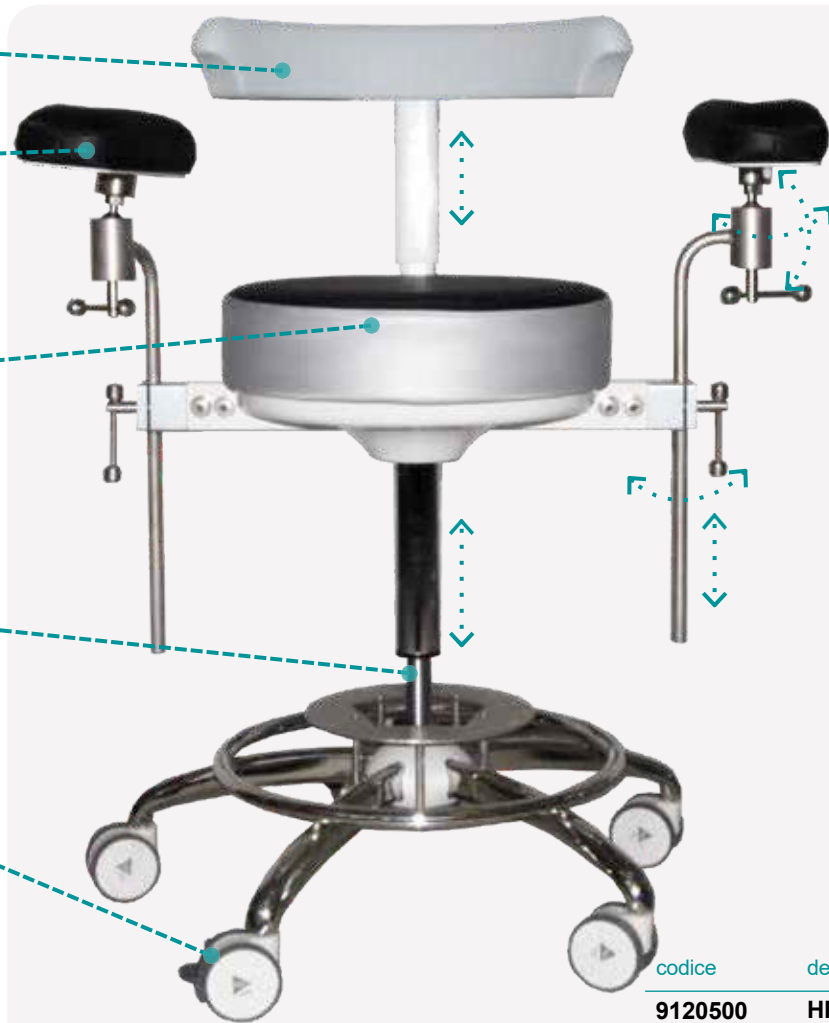
OPZIONALE

80 colours available

codice	descrizione
9120500	HP GEL - S (without foot-rest)
9120506	HP GEL - S (with foot-rest)
9120502	HP GEL - S HIGH (without foot-rest)
9120507	HP GEL - S HIGH (with foot-rest)

80 colours available

codice	descrizione
9120133	HP GEL WITHOUT FOOT REST
9120134	HP GEL WITH FOOT REST
9120136	HP GEL WITHOUT FOOT REST - HIGH
9120137	HP GEL WITH FOOT REST - HIGH

The seat naturally adapts to the user's body by increasing the contact area and decreasing the pressure on it. Circular seat with visco-elastic foam in different densities and memory effect. Multifunction armrest/back rest in high resistant polyurethane. Base in steel and polished chrome. 5 self braking wheels. Adjustable height by gas piston. High quality skay cover.

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OPZIONALE

EDGE SURGICAL SOLUTIONS develops and manufactures cabinets and furniture for: Ambulatory Surgery Center, Sterilisation Room, Outpatient Room and Front Office. The EDGE SURGICAL SOLUTIONS Furniture Project is paramount thanks to its innovative constructive and installation solutions that allow for an easier operating activity in the room, speed up the picking of needed instruments and material, guarantee the use of single modules as workstations during the procedures reducing obstacles and inefficiencies while allowing for a faster and easier disinfection of the operating area at the end of the activity when all modules can be stored back in the Corian® Garage. It's a space saving, clean and elegant setup that can be adapted to any room size.

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EDGE SURGICAL SOLUTIONS cabinets are entirely built in metal sheet, riveted and painted with epoxy powder, the drawers modules are independent and mobile thanks to the 4 medical grade castors with 2 front brakes and no-mark rubber; the drawers are finished with metal front panels, different dimensions and colours at choice, push-pull opening/closing system and handles/free for a better and easier disinfection. The Sink is built in the Corian® working top and it guarantees perfect disinfection. State of the art design, elegant and high quality material, perfect ergonomics and absolute infection control define this EDGE SURGICAL SOLUTIONS unique project as a benchmark in Medical Furniture.

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Garage Structure in steel and completely covered in Corian® to house the mobile modules. Splash-proof Corian® worktop guarantees easy cleaning and disinfection and long durability.

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Mobile Cabinet Modules, 4 medical-grade, non-marking wheels equipped with two front brakes for easy maneuverability.

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Drawers with push-pull system with stabilizer bar. Internal abs instruments trays for perfect disinfection and perfect storage.

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Corian® Worktop for an easier maneuverability of the mobile modules.

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Sink integrated on the Corian® worktop, available in both stainless steel and Corian® in different sizes and models.

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Hanging cabinets with drawers and flaps in different sizes and wall furniture with custom-made shelves.

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Possibility to integrate an archive and reporting desk with Methacrylate divider from operating area.

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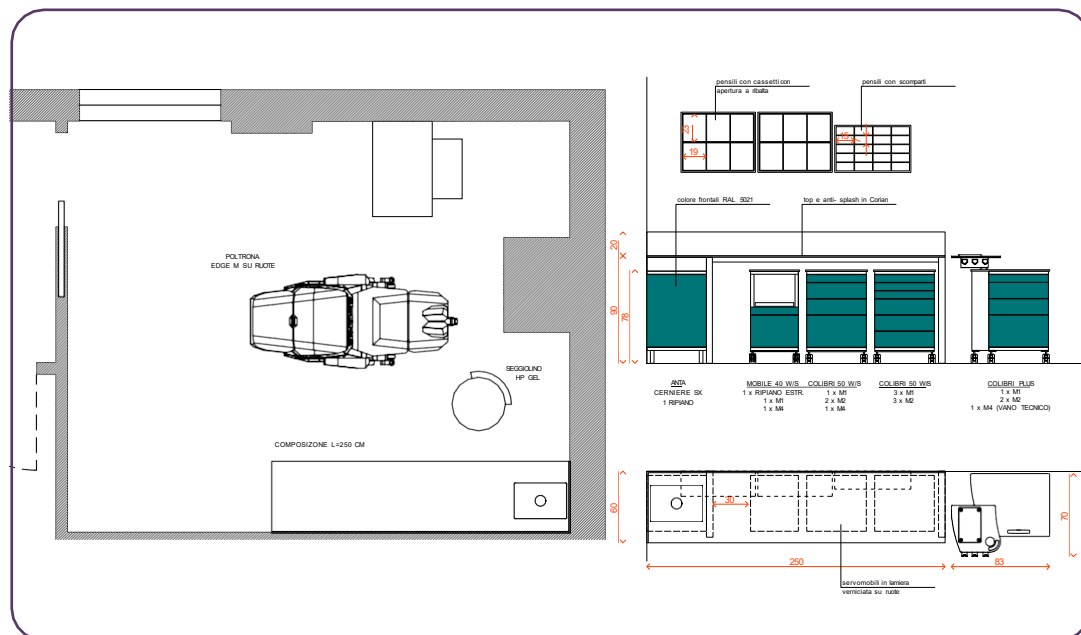
Possibility of integrating a stainless steel insert on the Corian® worktop to place the hot trays taken out from the autoclave.

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FURNITURE - MEUBLES - ARREDI

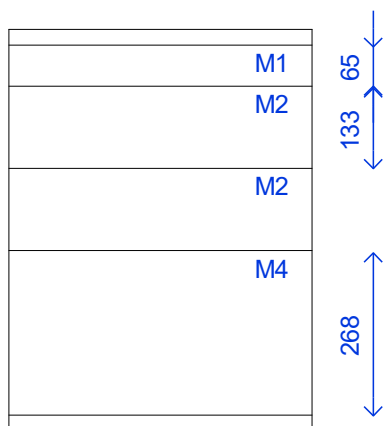
Corian® - Technical Sheet

- Corian® is a non-porous and homogeneous material, composed of approximately 1/3 acrylic resin and approximately 2/3 natural mineral substances.
- The junction points are imperceptible.
- Corian® surfaces are renewable (it can be fully restored to its original appearance by means of common detergents and an abrasive sponge).
- Corian® surfaces are hygienic.
- Corian® is an inert and non-toxic material.
- Corian® meets or exceeds emission guidelines for Volatile Organic Compounds (COV), Hazardous Air Pollutants (HAP).
- Chemical resistance. Class 1 reagents do not alter the material.

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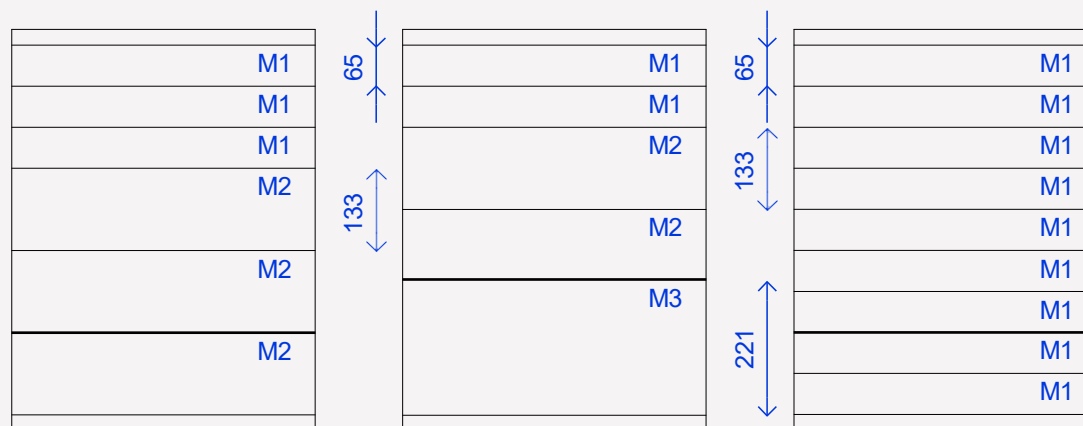
50 cm wide Cabinet with 4 drawers and metal fronts without handles provided with a Push-Pull opening system. Inside the drawers, storage trays for instruments. Cabinet on castors (front wheels equipped with brake) Worktop in Corian®.

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Cabinet 50 cm width. Standard configuration with 4 drawers with metal fronts without handles and Push-Pull opening system. Equipped with internal instrument trays. Corian® side compartment for the equipment lodging (load capacity 8 kg). Mobile on wheels for a comfortable handling (the two front wheels are equipped with brakes). Corian® worktop.

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Cabinet 85 cm width with 8 drawers (2 M1, 4 M2 e 2 M4) with metal fronts without handles and Push-Pull opening system. Equipped with internal instrument trays. Mobile on wheels for a comfortable handling (the two front wheels are equipped with brakes). Corian® worktop.

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Cabinet 90 cm width with 6 drawers (2 drawers 40 cm wide and 4 drawers 50 cm wide) with metal fronts without handles and Push-Pull opening system. Extendable metal shelf with Soft Touch system. Equipped with internal instrument trays. Mobile on wheels for a comfortable handling (the two front wheels are equipped with brakes). Corian® worktop.

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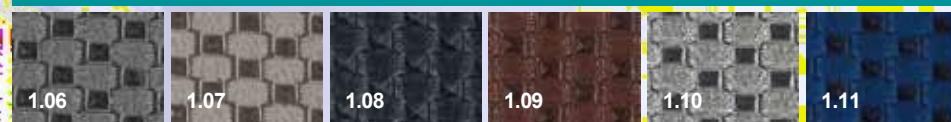
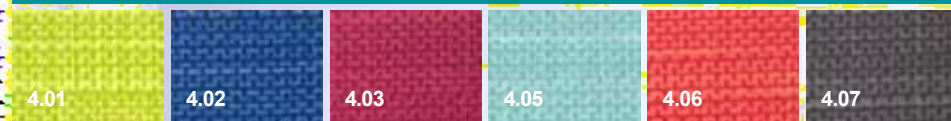
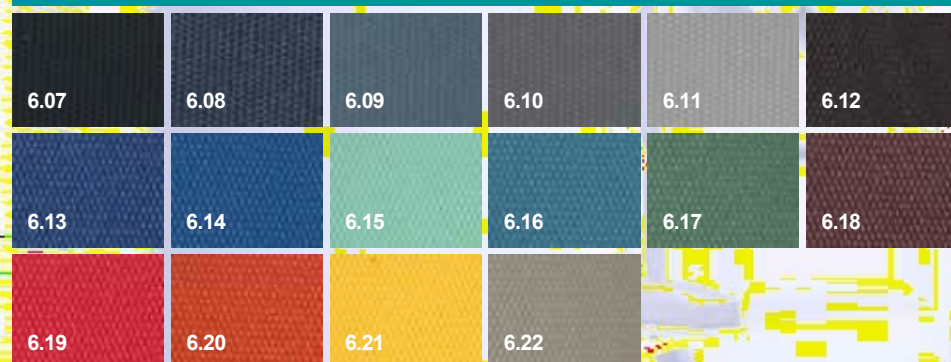
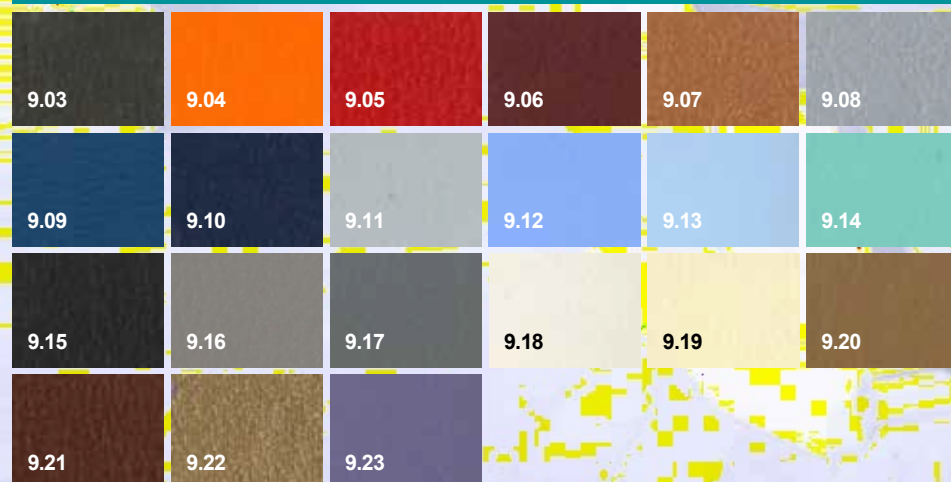
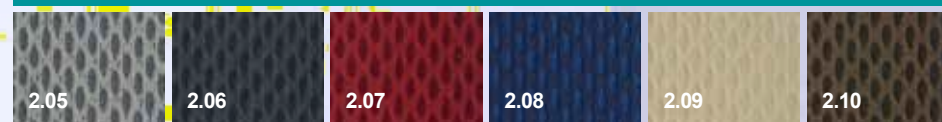
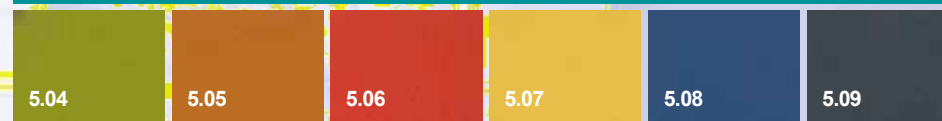
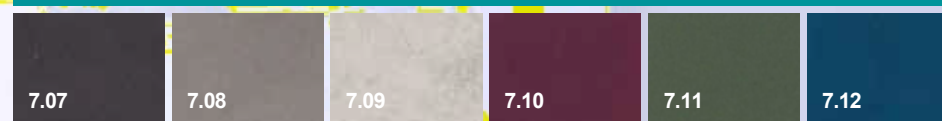
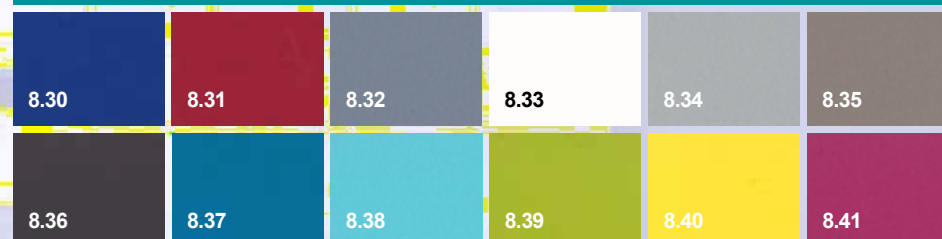
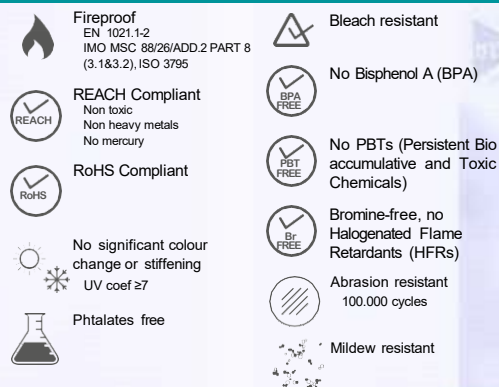
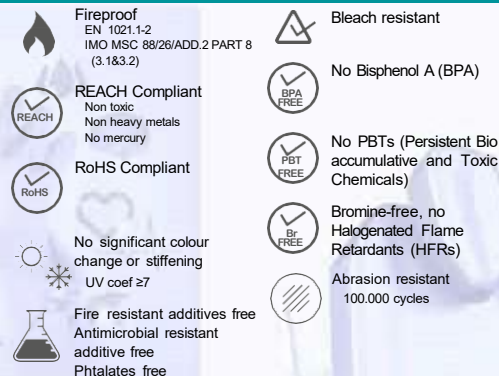
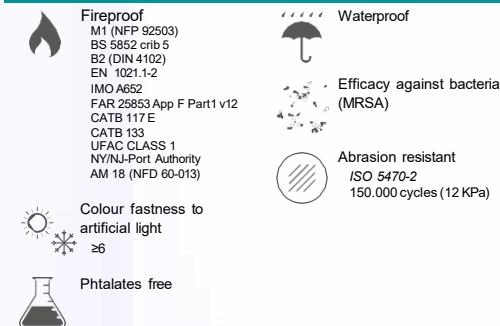
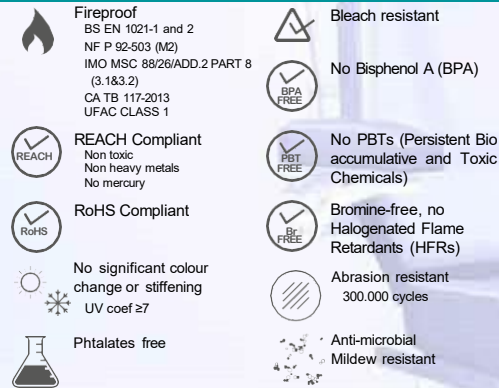
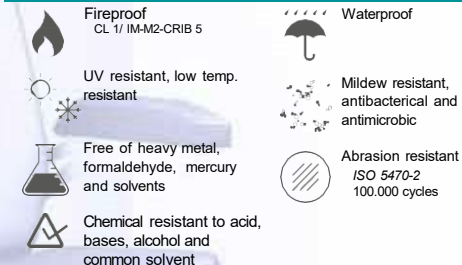
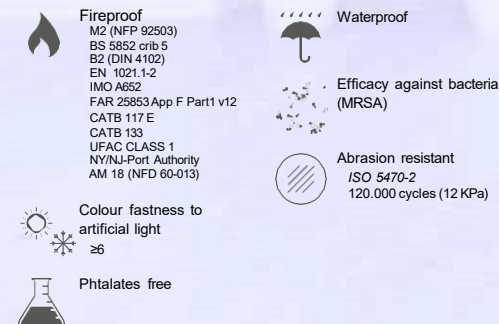
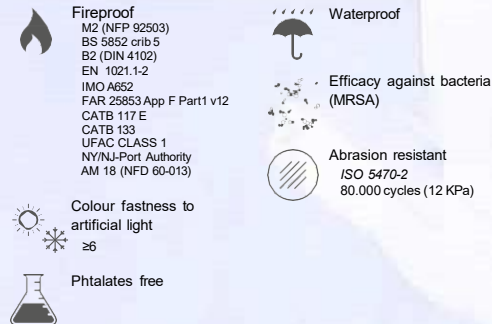
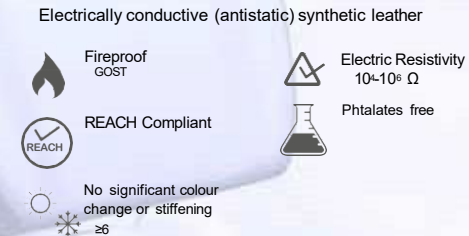
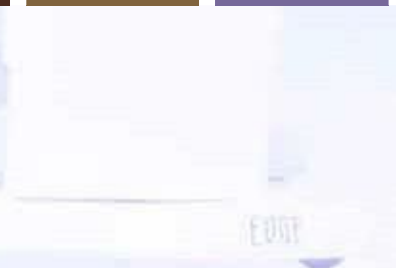
[illegible]

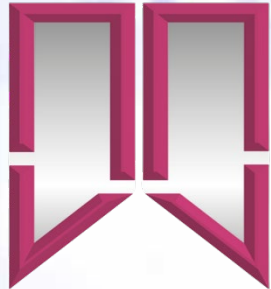
This image shows a full page of white paper with horizontal dotted lines. The lines are evenly spaced and run across the width of the page, providing a guide for handwriting practice. There are no margins, text, or other markings on the page.

www.edgesurgicalsolutions.com

edgesurgicalsolutions@gmail.com

*"ALLIEDGE SURGICAL SOLUTIONS Medical Equipment is designed ,
developed and manufactured in Italy"*

EDGE SURGICAL SOLUTIONS Srl

Headquarter: Via Umberto I, 48 73039 Tricase (LE) –ITALY

Production Site: S.S.275 Alessano-Lucugnano KM 17,5
73031 ALESSANO (LE) - ITALY